



SureStep™ Urine Drug Screen Test



Multi-Drug Test panel with
Integrated E-Z Split Key Cup
**A complete urine drug testing
system in a sealed cup**

Opiates

Cocaine

Benzodiazepines

Cannabis

Amphetamine

Methamphetamine

Multi-Drug Test Panel with Integrated E-Z split Key

Secure:

Eliminates exposure to urine and provides temperature verification to confirm urine samples. Inbuilt adulteration strip.

Convenient:

Simple, easy to use procedure – collect the urine, push the key and read results at 5 minutes. Tests for 6 drugs in the one device.

Stable:

Long shelf life stored at room temperature or when refrigerated (2-30°C).

Performance:

Accurate with high level of correlation to GCMS reference methods.

Verified to AS/NZS4308:2008 Appendix B.

**Only applicable to 6 panel test.*

Instructions for Use

- 1) Bring pouch to room temperature before opening.
- 2) Collect urine specimen in the cup and secure cap.
- 3) Check temperature label up to 4 minutes after specimen collection.
- 4) Push key into socket of the cup to begin the drug screen test.
- 5) Peel off label.
- 6) Read results shown at 5 minutes.

"PRODUCT PROCEDURE NOTE: Refer to product insert for complete instructions, limitations and warnings."

Results Interpretation

A line must appear in the control line region (C) to indicate that the test has worked correctly.

Ordering information

IVDBD-167-19A	25 cups/box	6 analytes
---------------	-------------	------------



Alere

Australia
1800 622 642 Free call in AUS
+61 7 3363 7100 Office
+61 7 3363 7199 Fax
au.enquiries@alere.com

alere.com.au

Alere Ltd

New Zealand
0800 466 951 Free call in NZ
0800 466 951 Office
0800 466 958 Fax
nz.enquiries@alere.com

alere.co.nz